

EXION™ Fractional RF

EXION™ Clear RF

AFTERCARE

Get the Best Out of Your Treatment



Wait 6 hours before
washing the area



Stay
well-hydrated



Do not apply makeup
for 24-72 hours



Cold compress
to calm the skin



Avoid sun exposure
and use SPF 30+ daily



Avoid touching the area
with unwashed hands



#EXION | EXION.COM



What To Expect After the Therapy?

- **Redness, heat sensation**, mild swelling or sensitivity to touch are part of the skin's natural reaction.
- **Tiny scabs** may appear within 1-3 days and may last for several days. Faint grid marks may take a few weeks to dissipate. **Do not touch** or scratch them; **let them naturally flake off**.
- Rarely, small crusts, blisters, burns or an acne breakout can develop in the treated area, but they typically slough within one week.
- Over the next few months, the production of **new collagen will gradually improve** your skin's **texture and color**.

How To Support the Healing Process?

- Apply a **healing cream** (anti-inflammatory or with Panthenol) **regularly** until your skin is no longer sensitive.
- **Don't rub** the area with a towel for the first 48 hours.
- Make sure to **disinfect any jewelry** or accessories when putting them back on the treated area.

What Activities to Avoid?

- Avoid activities involving **sweating, rubbing, or extreme temperatures** (like sauna, swimming, gym) for **2-5 days** post-treatment.
- Be **cautious** when your **pets** are **near the treated area**.
- **Avoid** other skin procedures, such as **facials, peels, lasers, or waxing** in the treated area, as well as the use of **tanning beds**.

Skin Care Routine

- You can use **regular foamy soaps** or **micellar water 24 hours** after the therapy.
- **Wait 4-7 days** before using **scrub soaps, exfoliants, heavy creams** or **retinols**.
- **The sunblock** should be used **during the course of all treatments** and for **at least 1 month** after the last treatment.



#EXION | EXION.COM



Your results and skin reaction may vary. As with any medical procedure, ask your doctor if EXION™ is right for you. The treatment may involve risks of complications or injury from both known and unknown causes. If there is any indication of an unusual or untoward symptom such as infection, excessive swelling or undue pain, contact your doctor. © 2024 BTL Group of Companies.

785-75EXIFPEN102;Exion_LF_For-patients_EN102